PL

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EN

Safety Instructions and Warnings

This list of safety warnings has been prepared in accordance with **Regulation (EU) 2023/988 on General Product Safety (GPSR)**.

Its purpose is to **protect users from potential hazards** resulting from improper use of the device.

The warnings are written in a **clear and understandable manner** to ensure accessibility for a wide audience, including **elderly people and persons with limited mobility**.

Safe Usage Guidelines

Safe usage rules:

- **Never apply the electrodes to the skin while the power is on** – this may cause a sudden electric shock.
- **If the device malfunctions or causes discomfort** – stop using it immediately.
- **When disposing of the device, accessories, or packaging** (batteries, films, foams, boxes), **follow local regulations**.
- **Keep the device away from** corrosive substances, flammable or explosive materials, high temperatures, and humidity.
- **If the device becomes wet** – stop using it immediately.
- **After moving the device from a cold to a warm environment** – wait before switching it on.
- **Do not disinfect the device with steam or under pressure.**
- **Do not immerse the device in liquids.**
- **Clean the device using isopropyl alcohol and a soft cloth.**
- **Do not spray liquids directly onto the device.**
- **Do not place electrodes on metal objects.**
- **Store the device in a dry place** – moisture may cause corrosion or damage.

- **Do not use the device while sleeping.**
- **Do not use the device while driving or operating machinery.**
- **Do not modify or repair the device yourself.**
- **Check the condition of the electrodes before use** and ensure they are properly attached.
- **Do not leave the device unattended while charging.**
- **Keep the device out of reach of children.**

Warnings and Contraindications

Do not use the device in the following cases:

- **Heart diseases** – without prior medical consultation,
- **Presence of a pacemaker or defibrillator,**
- **Epilepsy or cardiac arrhythmia,**
- **High fever or chronic infections,**
- **Recent muscle injuries (first 7–10 days),**
- **High or low blood pressure,**
- **Menstruation, pregnancy, postpartum period,**
- **Breastfeeding,**
- **Abdominal or inguinal hernia,**
- **Tumours (benign or malignant),**
- **Wounds, skin diseases, swelling, phlebitis, thrombosis, varicose veins,**
- **Moist skin** (after bathing, showering, or sweating),
- **Tuberculosis,**
- **Presence of metallic objects in the body,**
- **Vascular insufficiency,**
- **Inflammation of the skin or soft tissues,**
- **Eczema or other dermatological conditions,**
- **Diabetes with neuropathy** – only after consultation with a doctor,
- **Metallic orthopaedic implants** – only after consultation with a doctor.

Prohibited Electrode Application Areas

- **Carotid sinus, neck, throat, and mouth areas** – risk of severe muscle contractions and breathing difficulties,
- **Eyes and their surrounding areas,**
- **Chest area near the heart** – may cause heart rhythm disturbances,
- **Brain area.**

Additional Contraindications

- **Do not use the device on children, infants, or newborns.**
- **In case of any doubt,** consult a doctor before using the electrostimulator.

The product bears the CE mark, confirming its compliance with EU safety and quality requirements. It is not a medical device under current regulations.