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Safety and Warnings

This list of safety warnings has been prepared based on **Regulation (EU) 2023/988 on General Product Safety (GPSR)**.

Its purpose is to protect users from potential risks resulting from improper use of the device.

The warnings are written in a clear and simple manner to ensure accessibility for a wide audience, including elderly individuals and persons with limited mobility.

Safe Use of Electrodes

The following rules are intended to ensure safe, effective, and hygienic use of electrodes.

Before each session, read the instructions carefully and exercise caution.

General Safety Precautions

- **Personal product** – electrodes are intended for personal use only. Sharing increases the risk of cross-contamination.
- **No modification** – do not cut, trim, or alter the structure of the electrodes. This may disrupt impulse conduction and cause burns.
- **Do not use during activity** – do not use the electrostimulator while driving, operating machinery, bathing, or swimming.
- **Children and adolescents** – keep the device out of reach of children; do not use on infants. Children under 16 years of age may use the device only under direct supervision of a qualified specialist.

Preparation and Application

- **Clean skin** – before each application, wash and thoroughly dry the area where the electrodes will be placed. Remove any make-up, oils, creams, or other contaminants.
- **Skin condition** – do not apply electrodes to wounds, irritations, rashes, or scars. If redness or itching occurs, stop using immediately and consult a doctor.
- **Safe activation** – the device must be completely turned off and unplugged before applying the electrodes.
- **Proper connection** – insert the leads gently according to the manufacturer's instructions. Do not use force.
- **Starting the session** – turn on the device only after the electrodes are securely attached. Increase the current gradually, only up to a comfortable level.

Usage Guidelines

- **Careful removal** – after finishing the session, switch off the device. Remove the electrodes slowly, peeling them parallel to the skin surface. Do not pull by the wires.
- **Spacing between electrodes** – maintain a minimum distance of 3 cm between electrodes placed on the same muscle group.
- **Time limitation** – a single session should not exceed 30 minutes per muscle group. The maximum total stimulation time per day is 2 hours.

Storage and Maintenance

- **Proper conditions** – store electrodes in a sealed package at a temperature of **5–27 °C** and relative humidity of **40–70%**. Avoid sunlight and moisture.
- **Do not expose to air** – keep electrodes protected from air exposure to prevent loss of adhesiveness.
- **Electrode replacement** – replace the electrodes when they lose adhesion, become mechanically damaged, or after approximately 20 uses.

Specific Contraindications

- **Electronic implants** – if you have a pacemaker, neurostimulator, or insulin pump, use electrodes only with your doctor's approval.
- **Pregnancy and breastfeeding** – use of electrostimulation during pregnancy or breastfeeding requires prior consultation with a physician.
- **Epilepsy and heart diseases** – use is permitted only with explicit medical consent. Avoid application in the head and neck areas.
- **Adverse reactions** – if you experience burning, tingling, pricking sensations, or a metallic taste in your mouth, stop the treatment immediately and contact medical personnel.
- **Diabetes** – especially with neuropathy – stimulation should only be used after consulting a doctor.
- **People with metallic orthopaedic implants** should consult a physician before use (e.g., screws, joint prostheses).

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