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# Elektrostymulator TENS EMS AS-979

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## User manual

# Electrostimulator TENS EMS AS-979

The AS-979 TENS EMS electrostimulator is a **portable therapeutic, rehabilitation and relaxation device** that **naturally relieves pain, inflammation and swelling**.

**Please read this user manual before use.**

**Dear Sir or Madam, thank you very much for purchasing our product.**

This manual has been developed for the AS-979 electrostimulator. It describes the electrostimulator, its application, menu, functions, technical specifications, method of use, troubleshooting, etc. The manual also describes safety procedures designed to protect both the user and the device itself. Detailed information can be found in the relevant sections.

**Please read this manual carefully before using the device.**

Failure to follow the instructions may result in damage to the equipment or personal injury. The manufacturer, importer, or distributor are not responsible for the safety, reliability, performance, personal injury, or damage to the device resulting from the user's failure to follow the instructions in this manual. Warranty service does not cover such defects. This product is designed for repeated use.

**Electrostimulation** is most commonly used in orthopaedics, neurology and rheumatology, as well as in medical rehabilitation for various degenerative diseases of the joints, bones and muscles. Muscle electrostimulation is an electrotherapeutic treatment, also known as electrogymnastics, which aims to elicit a motor response in a muscle or group of muscles.

**The aim of electrostimulation is to regenerate muscles** or muscle groups, increase their strength and endurance, increase or decrease muscle and joint tension, improve circulation, protect against atrophy by improving local circulation and increasing temperature, reduce swelling and reduce spasticity. Electrostimulation involves pulses applied using point electrodes. The aim of the treatment is to stimulate the muscles to contract (the muscles contract and relax in rhythm with the intermittent current).

1. Connect the electrostimulator to the power supply unit from the package or insert 4 x 1.5 V AA batteries.
2. Connect the electrodes to the device:
  - ⊖ connect 2 or 4 electrodes to channel 1,
  - ⊖ connect 4 electrodes or massage pads to channel 2.

### IMPORTANT!!!

For foot massage, use the massage pads by connecting them to the electrostimulator using a connection cable with two or four outputs (connect one pad to the white outputs and the other to the red outputs). In order for the electrostimulator to start working (transmitting TENS/EMS currents), attach the electrodes to the body or put on the flaps. **The device does not work without connecting the electrodes to the body.**

3. Turn on the device by pressing the black round button in the upper left part of the main device ON/OFF (hold for about 2-3 seconds).
4. Use the knobs located in the upper left part of the main unit (Channel 1) and the upper right part of the main unit (Channel 2) to independently control the two channels, setting the power of the device on a given channel.
5. Turn the power knob to a level that is noticeable (optimal) but not painful. During the treatment, you can adjust the power of the device by increasing or decreasing it according to your needs.

Over time, the body becomes accustomed to a given stimulation intensity – in such cases, it is recommended to increase the strength/intensity of the therapy.

6. The electrostimulator has the following operating modes located at the bottom of the device:

- ⊖ PRESS – pressing (TENS)
- ⊖ KNEAD – rubbing, kneading (TENS)
- ⊖ VIBRATE – vibration, pulsation (TENS)
- ⊖ THUMP – tapping (EMS)

7. The therapy for a specific part of the body can be adjusted by selecting the programs located on the left and right sides of the electrostimulator:

- ⊖ SHOULDER – shoulders/neck (TENS)
- ⊖ WAIST – waist/loins (TENS)
- ⊖ LEG – legs (thighs, calves) (TENS)
- ⊖ ARM – arms (TENS)
- ⊖ JOINT – joints/knees (TENS)
- ⊖ FEET – feet (TENS)

8. On the left side of the device there are two additional programs using EMS currents:

- ⊖ SLIMMING – a slimming and weight loss program that helps reduce body fat
- ⊖ FITNESS – a program that helps build muscle mass

9. After the treatment is complete, switch off the device using the ON/OFF button and then disconnect the electrodes from the body.

10. The electrodes should be protected with the original protective film or stuck to a double-sided pad.

### How to connect the electrodes to the AS-979 electrostimulator

For foot massage, use the slippers by connecting them to the electrostimulator using a connecting cable with two outputs (Fig. 1) or four outputs (Fig. 2). Connect one slipper using the WHITE outputs and the other slipper using the RED outputs.

**Fig. 1**

**Fig. 2**


Button ON/OFF

Power control knob (channel 1)

1) Program-ARM  
2) Program-JOINT  
3) Program-FEET

Weight loss (fat reduction)

Muscle sculpting / muscle building

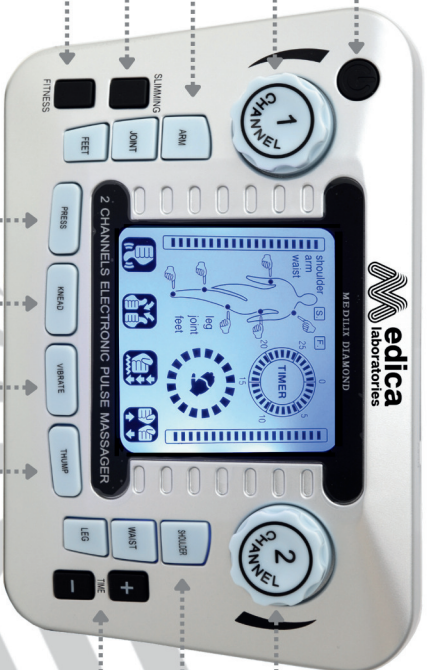Power control knob (channel 2)

1) Program-SHOULDER  
2) Program-WAIST  
3) Program-LEG

Reduction/increase  
In time

Mode

Mode

Mode

Mode

## Operational safety

- ⊗ Remove the batteries if the product will not be used for a long period of time.
- ⊗ Never attach the electrodes to your skin while the device is switched on – there is a risk of electric shock.
- ⊗ If the device malfunctions or causes discomfort, stop using it immediately.
- ⊗ Do not use the device in a damp environment, while bathing, driving or sleeping.
- ⊗ Service and maintenance may only be performed by qualified personnel. The user must not perform maintenance themselves.
- ⊗ Do not use the electrostimulator simultaneously with devices not described in this manual.
- ⊗ Do not use the device in a flammable environment – risk of explosion.
- ⊗ Dispose of the device, accessories, batteries and packaging in accordance with local regulations.
- ⊗ Keep the device away from corrosive substances, explosives, high temperatures and moisture.
- ⊗ If the electrostimulator or accessories get wet, stop using them immediately.
- ⊗ After moving the device from a cold and humid environment to a warm one, wait before turning it on.
- ⊗ Do not press the main panel with sharp objects.
- ⊗ Do not disinfect the device with hot steam or high pressure.
- ⊗ Do not immerse the electrostimulator in liquids – clean it with a soft cloth soaked in medical alcohol.
- ⊗ If you are storing the device for more than a month, remove the batteries to prevent them from leaking.
- ⊗ Do not connect the electrodes to metal devices.
- ⊗ Store the device in a dry place – excessive moisture may damage it.

## Warnings and contraindications

- ⊗ Heart disease without prior consultation with a doctor.
- ⊗ Persons with pacemakers or other implanted defibrillation devices.
- ⊗ Diseases such as epilepsy, arrhythmia.
- ⊗ High fever, chronic infectious diseases.
- ⊗ Early period after muscle injury (7–10 days).
- ⊗ Abnormal blood pressure.
- ⊗ Menstruation, pregnancy or postpartum period.
- ⊗ Breastfeeding.
- ⊗ Abdominal or inguinal hernia.
- ⊗ Benign and malignant tumours.
- ⊗ On cuts/skin diseases/skin lesions/oedema/phlebitis/thrombophlebitis/varicose veins.
- ⊗ If the body is wet after bathing, showering or as a result of sweating.
- ⊗ Tuberculosis.
- ⊗ Presence of metal foreign bodies in tissues.
- ⊗ Blood vessel insufficiency.
- ⊗ Inflammation of the skin and soft tissues.
- ⊗ Eczema (dermatological lesions).
- ⊗ Electrodes must not be placed in the carotid sinus area, neck, larynx, mouth and throat, or on the lips, as this may cause severe spasms of the larynx and throat muscles; these spasms may be severe enough to close the airways or cause breathing difficulties.
- ⊗ Do not place electrodes above or on the eyes.
- ⊗ Do not stimulate the chest in the area of the heart, as the introduction of electrical current into the heart may cause cardiac arrhythmia.
- ⊗ Stimulation must not be applied through the brain.


## Additional comments

- ⊗ The long-term effects of chronic electrical stimulation are unknown.
- ⊗ Do not use the device on children, infants or newborns.
- ⊗ The product is intended for adults.
- ⊗ If you are unsure whether you can use the electrostimulator, consult your doctor.

## Indications for use

- ⊗ body shaping
- ⊗ weight loss (fat reduction)
- ⊗ increasing muscle endurance
- ⊗ relaxation and unwinding
- ⊗ warm-up before training
- ⊗ muscle development (abdominal muscles, deltoids, biceps, triceps, forearm muscles, glutes, quadriceps, calf muscles)
- ⊗ prevention or inhibition of muscle spasms
- ⊗ re-education of lost muscle functions
- ⊗ maintenance or increase of range of motion in joints
- ⊗ rehabilitation after injuries
- ⊗ foot drop
- ⊗ urinary incontinence – stimulation of the peroneal nerve
- ⊗ back pain (cervical, shoulder, thoracic, lumbar)
- ⊗ torticollis (stiffness/spasm of the neck)
- ⊗ improvement of local blood circulation
- ⊗ pain after mechanical injuries
- ⊗ degenerative conditions
- ⊗ neuralgia
- ⊗ rheumatism
- ⊗ migraines
- ⊗ tennis elbow syndrome
- ⊗ elbow pain
- ⊗ knee pain

Cable with 4 outputs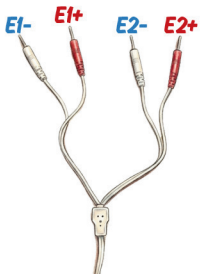

Cable with 2 outputs


TENS


Chronic cervical spine pain

∅ PROGRAM: SHOULDER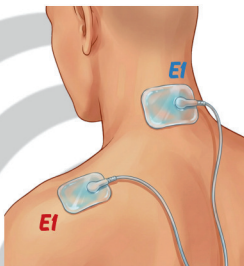

Wrist pain

∅ PROGRAM: ARM

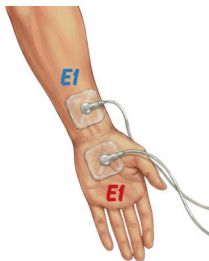
Ankle pain

∅ PROGRAM: FEET


Relieving stomach ache

∅ PROGRAM: WAIST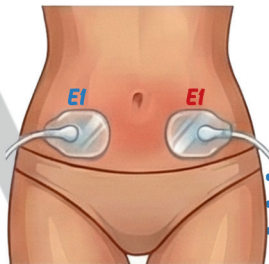

## Elbow joint pain


PROGRAM: **ARM**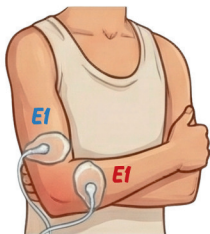

## Knee pain

Osteoarthritis of the knee


PROGRAM: **JOINT**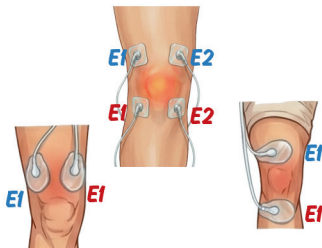

## Shoulder pain

Compression of the spinal cord in the cervical region


PROGRAM: **SHOULDER**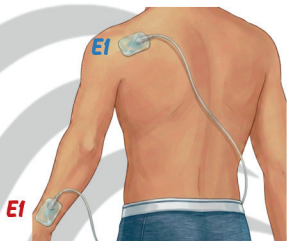

## Excessive tension in the paraspinal muscles


Thoracic section

PROGRAM: **SHOULDER**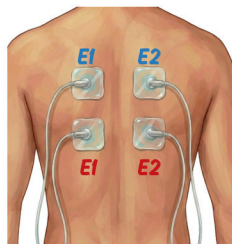


## Sciatica

PROGRAM: **WAIST**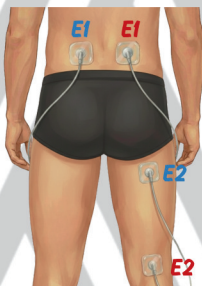


## Shoulder pain

PROGRAM: **ARM**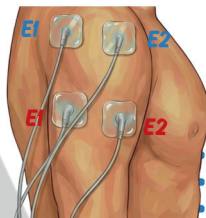


### Pain or tension in the calf

Ø PROGRAM: **LEG**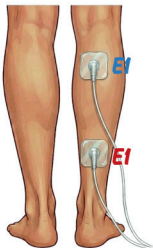


### Ankle pain

Ø PROGRAM: **FEET**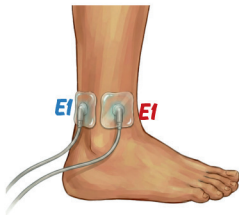

### Shoulder blade pain


Ø PROGRAM: **SHOULDER**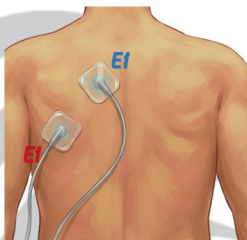

### Lumbosacral pain

Ø Tension in the lumbar spine muscles without pain radiating to the lower limbs


Ø PROGRAM: **WAIST**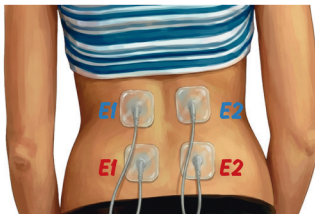

### Tension in the paraspinal muscles


Ø Cervical section

Ø PROGRAM: **SHOULDER**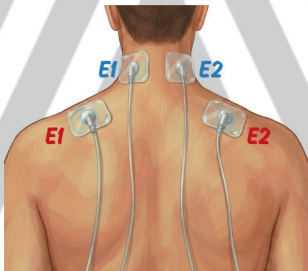


### Hip pain

Ø PROGRAM: **LEG**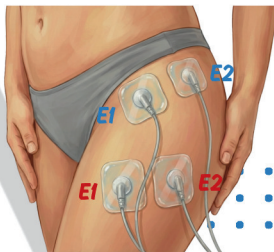


### Urinary incontinence

- ∅ Electrostimulation of the tibial nerve (short and long) (peroneal)
- ∅ PROGRAM: **LEG**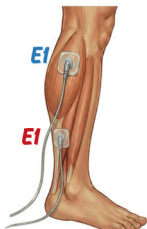

### Falling foot

- ∅ Damage to the nerve or atrophy of the tibialis anterior muscle
- ∅ PROGRAM: **LEG**

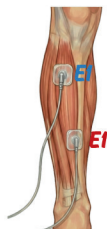
EMS


### Gluteal muscles

- ∅ PROGRAM: **FITNESS/SLIMMING**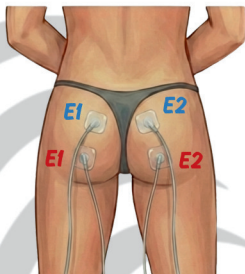


### Quadriceps muscle of the thigh

- ∅ PROGRAM: **FITNESS/SLIMMING**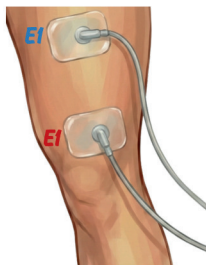


### Abdominal muscles

- ∅ PROGRAM: **FITNESS/SLIMMING**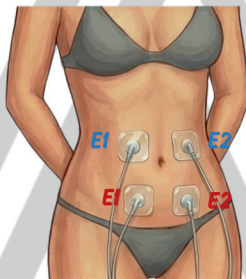


### Deltoid muscle

- ∅ PROGRAM: **FITNESS/SLIMMING**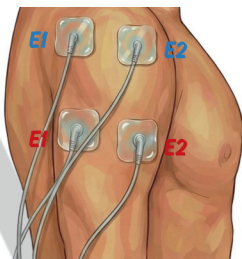

### Arm muscles

PROGRAM: FITNESS/SLIMMING

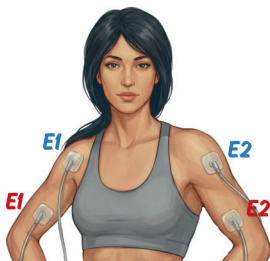
### Forearm muscles

PROGRAM: FITNESS/SLIMMING

### Calf muscles

PROGRAM: FITNESS/SLIMMING

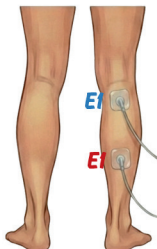### Biceps muscle

PROGRAM: FITNESS/SLIMMING

### ATTENTION!

Electrotherapy does not replace medical treatment.

Before using the electrostimulator, it is recommended to consult a doctor or physiotherapist, especially in the case of chronic diseases, pain of unknown origin, after injuries or surgical procedures.

The user uses the device at their own risk and is obliged to follow the safety rules and recommendations contained in the user manual. The manufacturer, importer or distributor are not liable for the consequences of improper use of the device or use that is not in accordance with the instructions.

## How to care for electrodes to extend their service life

- ⊗ Ensure that the device is switched off.
- ⊗ If necessary, remove any hair from the application site.
- ⊗ Degrease and dry the skin (this extends the life of the electrodes).
- ⊗ Connect the electrodes to the device and remove the protective film (keep it for reuse; if damaged, use the double-sided pads from the kit).
- ⊗ Stick the electrodes to the skin and press lightly.
- ⊗ Read the device manual before use.
- ⊗ After the treatment, slowly peel off the electrodes, starting from the corner, and stick them back onto the protective film.
- ⊗ Store the electrodes in their packaging in a dry place.
- ⊗ Replace the electrodes when they stick poorly or the gel is dry or dirty.
- ⊗ Do not stick exposed electrodes together as this may damage them.
- ⊗ The electrodes are not intended for disinfection; each user should have their own.

## Technical specifications of the device

- 2 stimulation techniques: EMS and TENS
- 8 programmes + 4 therapeutic modes
- power adjustment
- operating time adjustment
- 2 channels – independently adjustable power (possibility of stimulating two people at the same time)
- possibility of connecting 2 electrodes, 4 electrodes, 6 electrodes, 8 electrodes, 2 flap electrodes, 4 flap electrodes
- large, easy-to-read LCD display
- therapy time display
- battery or mains power supply – AA batteries not included
- frequency 0.5–150 Hz
- effective power (max): 200mA
- pulse width: 70–500  $\mu$ s
- mobile device
- CE marked product

## Problem solving

### There is no stimulation.

- ⊗ Check that the electrodes and cable are connected correctly.
- ⊗ Ensure that the electrodes are firmly attached to the skin.
- ⊗ Check that the batteries are not flat; replace them if necessary.
- ⊗ Ensure that the batteries are inserted correctly (+/-).

### The display and therapy mode are illuminated, but no stimulation occurs.

- ⊗ Check that the electrodes are positioned correctly and adhere well to the skin (the device may switch off after approx. 10 seconds if an electrode falls off or if not all of the two or four electrodes are attached to the body).
- ⊗ Check the power level and adjust the intensity so that you feel a gentle stimulation.
- ⊗ Ensure that the electrodes do not overlap.

### Poor stimulation

- ⊗ Ensure that the electrodes do not overlap.
- ⊗ Check that the electrodes and cable are connected correctly.
- ⊗ Check the correct placement of the electrodes.
- ⊗ Ensure that the electrodes adhere well to the skin and are not dry; replace them with new ones if necessary.
- ⊗ Check that the stimulation power level is not set too low.

### When switched on, neither the display nor the therapy modes light up.

- ⊗ Check that the batteries are not flat and replace them with new ones if necessary.
- ⊗ Ensure that the batteries are inserted correctly (+/-).

## Problem solving

### The skin becomes reddened.

- ⊘ Check the therapy time; do not exceed 10–15 minutes.
- ⊘ Make sure that the electrodes are not dry or dirty; replace them with new ones if necessary.
- ⊘ Check that the electrodes adhere well to the skin.

### One electrode works more strongly than the other.

- ⊘ This is normal, different parts of the body react differently.
- ⊘ Make sure that the electrodes are not dry and that they adhere well to the skin.

### Gel electrodes do not stick to the skin even after cleaning and moistening.

- ⊘ The self-adhesive gel pads should be replaced.

## Accessories

- ⊘ body electrodes (figure-eight shape)
- ⊘ body electrode (rectangle) – OPTIONAL
- ⊘ body electrode (butterfly) – OPTIONAL
- ⊘ body electrodes (square) – OPTIONAL
- ⊘ massage pad electrodes (function as foot electrodes)
- ⊘ glove electrodes with connecting cables – OPTIONAL
- ⊘ sock electrodes with connecting cables – OPTIONAL
- ⊘ knee electrodes with connecting cables – OPTIONAL
- ⊘ 240 V mains power supply
- ⊘ connecting cable for 4 electrodes
- ⊘ connecting cable for 2 electrodes
- ⊘ double-sided pad
- ⊘ case
- ⊘ cardboard box
- ⊘ user manual

The **AS-979** electrostimulator is a product that meets all safety standards. It has **two CE certificates (EMC and LVD)**. The **EMC** certificate complies with Directive 2014/30/EU of the European Parliament and of the Council of 26 February 2014 on electromagnetic compatibility. The **LVD** certificate complies with Directive **2014/35/EU** of the European Parliament and of the Council of 26 February 2014 – Low Voltage Directive.

This symbol indicates that this product must not be disposed of with other household waste. The user is responsible for delivering the used device to a collection point for the disposal of electrical and electronic equipment. Separate collection and recycling of devices helps to protect the environment and human health. For more information on the disposal of devices, please contact your local authorities, disposal companies, or the place where you purchased this product.

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